

TECHNICAL DATA SHEET

SteriCleen Neutral Enzymatic Detergent

CODE: 9225

Australian Medical Device Class 1
ARTG No. 332455

DESCRIPTION:

SteriCleen Neutral Enzymatic Detergent is a highly active neutral enzyme-based medical device detergent specifically formulated for the removal of hospital and surgical soils such as blood, body tissue, and other protein-based soils via both machine and manual cleaning operations.

SteriCleen Neutral Enzymatic Detergent is a concentrated formula which is low-foaming, non-staining and effective in both hard and soft water.

BENEFITS:

- Concentrated blend with neutral pH– safe for use on most instruments and devices
- Biodegradable surfactant system with recyclable packaging
- Acts quickly on blood and other protein-based soils
- Non-toxic, non-abrasive, low foaming, free rinsing
- Contains corrosion inhibitors for the prevention of corrosion of medical instruments during the cleaning process
- Can be used in Ultrasonic & Manual cleaning of instruments prior to disinfecting or sterilisation procedures
- Complies with AS/NZS4187 and AS/NZS4815

INTENDED USAGE:

- Use as a dedicated cleaner in conjunction with Ultrasonic cleaning machines.
- Manual cleaning of Medical and Dental instruments prior to the disinfection or sterilisation process.
- Manual cleaning of theatre floors, tables, benches as well as laboratory glassware.
- Manual cleaning of large and small equipment
- Do not allow biological contaminants to dry. If Contaminants have dried, soak at least for 20 mins at a dilution of 1:50

DIRECTIONS FOR USE:


1. Manufacturer's instructions should be strictly followed as advised by the instrument provider.

2. Prepare a fresh ready to use solution daily.

3. Dilution Instructions:

Soil and contamination level	Dilution Rate	Recommended soak time
HEAVY	1:100 (10ml per litre)	7 – 9 minutes
MEDIUM	1:150 (7.5ml per litre)	7 minutes
LIGHT	1:200 (5ml per litre)	5 minutes

Use warm water to dilute to achieve optimum performance (30° C - 40° C).

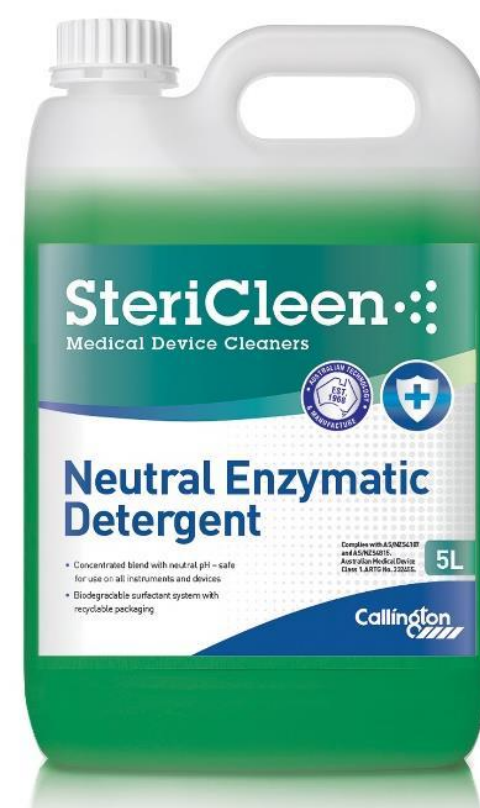

TECHNICAL DATA SHEET

Apply longer soak times if residue is dried on instrument.

Rinse with water after removal from soak solution and prior to sterilisation / disinfection.

4. Discard solution after use

Note: Instrument cleaners are not disinfectants, and all instruments should be cleaned, rinsed and drained before immersion in an instrument grade DISINFECTANT, or prior to autoclaving.

CHEMICAL COMPOSITION AND PROPERTIES:

Colour concentrate:	Dark Green
Colour dilute solution:	Colourless
Odour (Concentrate):	no odour
pH concentrate:	7.0-9.0
pH dilution:	7.5 ± 0.5
Solubility:	Completely soluble in water
Rinsibility:	Residue free
Specific gravity:	1.02 ± 0.05g/ml
Corrosion inhibitor:	Proprietary blend
Shelf life:	2 years

STORAGE CONDITIONS: Store below 25°C.

SAFETY DATA: Information is listed in the Material Safety Data Sheet.

PACK SIZES: 500mL CODE: 9225/25
5 Litres CODE: 9225/42

WARRANTY – All statements, information and data presented herein are believed to be accurate and reliable but are not to be taken as a guarantee, expressed or implied, for which seller assumes legal responsibility and they are offered solely for your consideration, investigation and verification. Statements or suggestions concerning possible use of this product are made without representation or warranty that any such use is free of patent infringement and are not recommendations to infringe on any patent. Created 30 November 2021 Date Printed 15/02/2022 2:40 PM